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**ENGLISH - DIRECTIONS FOR USE**

PRODUCT NAME: ECG/EEG gel/spray

PRODUCT DESCRIPTION: Electrode conductive medium

TRADE MARK: TRANSOUND®

INTENDED USE: ECG spray: Applied to clean solvent, cream or lotion sort substances off on ECG instrument. The spray provides the electronic information to access the ECG machine properly. Prolongs electrode life by removing bacteria and wastes cumulated on the surface of electrode's surface. The product increases conductivity. ECG gel: It is used under ECG electrodes in ECG imaging. Thanks to the active substances it contains, it increases the conductivity of the electrode.

COLOUR: colorless

PACKAGING

61.250.02 - ECG/EEG gel - 250 ml bottle, pack of 25 bottles

61.250.19 - ECG/EEG spray - 250 ml bottle, pack of 25 bottles

61.005.08 - ECG/EEG gel - 5000 ml bag, pack of 2 pcs. complete with 2 250 ml bottles (empty) for filling and nozzle.

61.005.09 – ECG/EEG spray - 5000 ml jerry-can, pack of 4 pcs. complete with 4 250 ml bottles (empty) + 1 pump for filling

OPERATION PRINCIPLE

ECG/EEG spray: is used to purify solvents, creams and lotions. It ensures that the information is clear to the device. It is an economical product. An average bottle can be used a minimum of 1500 times. It gives more effective results than cleaning with an ordinary product. It is used externally without dilution and provides hygiene. With the chemicals in its composition, it removes the wastes accumulated on the surface of the ECG electrode, extending the life of the electrode, and ensures longer use of the electrode. It increases conductivity. ECG/EEG gel: is used in all kinds of ECG and effort applications, non-gel electrodes, ECG clamps and poles. The gel ensures a clear transfer of ECG waves to the device.

CHARACTERISTICS

ECG Spray is hypoallergenic. It does not contain oil, phenol, aldehyde etc. It does not contain harmful toxic substances. It does not cause irritation. ECG gel is hypoallergenic, does not irritate the skin, does not contain oil and oily substances, formaldehyde and salt. It is odorless and has no toxic effect.

WARNINGS

- Keep between -10 °C and 50 °C
- Keep away from direct sunlight
- Keep dry
- This product can be used only by professional users.
- This product cannot be used on patients with known sensitivity to Monopropylene Glycol.

CONTRAINDICATIONS: N.A.

APPLICATION

It is used by applying a sufficient amount on the area or skin to be treated. Follow the instructions for use of the product and continue to apply the gel throughout the ECG treatment. After the treatment is completed, clean the ultrasound gel from the device and the applied area with the help of a cloth or towel. Immediately after use, wipe and clean the tools on which the gel is applied.

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