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## ENGLISH - DIRECTIONS FOR USE

**PRODUCT NAME:** Ultrasound gel

**PRODUCT DESCRIPTION:** Contact medium for ultrasound transmission

**TRADE MARK:** TRANSOUND®

**INTENDED USE:** Gel can be used as contact media in non-invasive medical procedures (diagnostic or therapeutic) that require viscous gel.

**COLOUR:** blu/transparent

### PACKAGING

61.250.01/61.250.11 - 250 ml bottle, pack of 25 bottles

61.001.01/61.001.03 - 1000 ml bottle, pack of 10 bottles

61.005.01/61.005.12 - 5000 ml bag, pack of 2 pcs. complete with 2 250 ml bottles (empty) for filling and nozzle.

61.005.02/61.005.13 - 5000 ml jerry-can, pack of 4 pcs. complete with 4 250 ml bottles (empty) + 1 pump for filling

### OPERATION PRINCIPLE

In all ultrasonography and doppler applications, it eliminates the air between the skin and the probe, providing better penetration of sonography waves into the device examination area and reducing losses in reflected waves.

### CHARACTERISTICS

The product is non-toxic, non-irritating, non-staining, water soluble, paraben free, salt free, PVC free, hypo-allergenic, dermatologically tested and does not contain formaldehyde. It is compatible with PE/PU/ Polyisoprene/Latex covers for ultrasound probes/transducers.

### WARNINGS

- For medical and professional use only
- Do not swallow
- Follow the portioning instructions
- Do not keep the lids, covers, caps, flaps open
- Skin disinfectant may be applicable onto examination area before and after the application.
- Stop using the gel for hypersensitive patients if skin reactions occur.
- Clean the transducer with clean wipe and disinfect it according to the manufacturer's recommendations.

**CONTRAINDICATIONS:** N.A.

### APPLICATION

- Apply a sufficient quantity of gel on the transducer or the patient's intact skin.
- Follow the instructions of transducer manufacturer's instructions.
- It is recommended to use a cover for the ultrasound probe/transducer.

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